435 | IsoCoat Zytex® | Gebrauchsanweisung / Instructions for Use

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Manufacturer's Instructions and Information

Please read this user information carefully before using this product. This product complies with the requirements of EU Regulation (EU) 2016/425 for Personal Protective Equipment and meets the requirements of European standard:

- Type PB [6]: EN 13034:2005+A1:2009 in conjunction with EN 14325:2018
- Type PB [6]-B: EN 13034:2005+A1:2009 in conjunction with EN 14325:2018, & EN 14126:2003/AC:2004

Products Name: Protective clothing

Products Colour: yellow

Products materials: 100% polypropylene nonwoven fabric coated with polyethylene film, 50 gsm

Brand name: IsoCoat Zytex®

Product reference: 435

Intended use of this PPE

The protective gowns are typically used, depending on chemical toxicity and exposure conditions, for partial body protection against low concentration inorganic fine sprays of water-based chemicals and infective agents (Type PB [6], Type PB [6]-B). Fabric used for the protective clothing has been tested according to EN14126 (protective clothing against infective agents) with the conclusion that the material offers a limited barrier against infective agents. Do not use out of the scope of use defined in the warnings. Failure to properly use this product may result in serious health damage or death. Stitched seams are not a barrier to infective agents.

This product is designed to protect against the risks:

Risk: Resistance To infective agents

Standard Clause: EN 14126:2003/ AC:2004 clause 4.1.4

Assessment method:

- Resistance to penetration by contaminated liquids under hydrostatic pressure
- Resistance to penetration by infective agents due to mechanical contact with substances containing contaminated liquids
- Resistance to penetration by contaminated liquid aerosols
- Resistance to penetration by contaminated solid particles

Check before using

1. Please read the users instruction carefully before using. 2. It is recommended that prior to use, full training is given on the safe use and limitations by a competent person with details of the training recorded. 3. Please select and confirm the protective clothing suitable for your size before using. 4. Do not use if package is damaged, discoloration, pollution, etc. 5. Before using, please ensure that all seams and closures are intact. 6. Check that the expiry date has not been reached (see the label).

Preparing for use: In the unlikely event of defects, do not wear the garment. Return the defective garment (unused and uncontaminated) to your distributor.

Warning

1. Prior to use, review all instructions and inspect the clothing for any damage that could affect its protective function (e. g. holes, damaged seams and fastenings, heavily soiled areas). Replace any damaged clothing. 2. Flammable material. Keep away from fire. 3. Care should be taken when removing contaminated garments, so as not to contaminate the user with any hazardous substances. If garments are contaminated, then decontamination procedures should be followed (e. g. decontamination shower) prior to removal of the garment. 4. Failure to follow all instructions and limitations on the use of this product, or failure to achieve proper fit, may result in damage to your health or death or will not provide the expected level of protection. 5. Do not use when concentrations of contaminants are immediately dangerous to health or life. Do not use this product in an explosive atmosphere. 6. Upon contamination, wear or damage the garment should be removed and disposed of properly. 7. Wearing the clothing may cause reduction of the field of vision, acuity of hearing or a risk of heat stress, wearing absorbent undergarments is recommended. 8. The user must ensure suitable reagent to garment compatibility before use. 9. Type PB [6] partial body protection has not been tested to the whole suit test. 10. Unsuitable for use with organic chemicals.

Storage and Transportation

Do not store in excess heat or direct sunlight. The protective clothing may be stored between +0 °C and +70 °C with relative humidity no more than 80%, with no UV light exposure and no corrosive gas and well-ventilated environment. Product shall be transported and stored in its original packaging.

Disposal and shelf life

The protective clothing can be incinerated or buried in a controlled landfill without harming the environment. Disposal of contaminated garments is regulated by national or local laws

Shelf life: 5 years. Date of manufacture is shown on the product label.

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Notified body

EU type-examination and conformity to type certifications module D for Regulation (EU) 2016/425 by Notified Body Number: 0598, SGS Fimko Oy, Takomotie 8, FI-00380 Helsinki, Finland.

EU Declaration of Conformity

EU declaration of conformity can be accessed at:
dach-germany.de/downloads

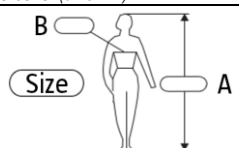
dach-germany.de/downloads

Manufacturer

DACH Schutzbekleidung GmbH & Co. KG
Address: Rotackerstr. 21, 76437 Rastatt, Germany
Detailed content of the Instructions and information notice is the responsibility for manufacturer.
Made in China.

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1. First, put on the work clothes, hats, etc. neatly, roll the sleeves over the elbow; take the clothes in hand to get the protective clothes.

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2. Hold the gown in your hand and fold the two ends of the neck opening out, shoulder to shoulder, to reveal the inside of the sleeve. Easy to wear.

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3. Hold the neck opening in one hand, put the other hand into the sleeve, raise the arm, put on one sleeve, hold the neck opening in the other hand, and put on the other sleeve in the same way.

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4. Hold the neck opening in both hands, straighten out the neck opening edge from front to back, and fasten the neck ties with a loop.

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5. Pull the back of the gown forward about 5 cm below the seam of one side. When you see the edge of the garment, hold it, and then hold the other side in the same way.

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6. With both hands at the back, align the edges, fold to one side, press and hold the fold, cross the belt at the back or left with a loop.

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Follow the step below to take off: 1. Untie the waist belt and neck tie loop and take off the protective gown. 2. Disposal of contaminated garments is regulated by national or local laws. Do not litter contaminated protective gown.

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1. Classification of abrasion resistance: Class 1 >10rubs; Class 2 >40rubs; Class 3 >100rubs; Class 4 >400rubs; Class 5 >1000rubs; Class 6 >2000rubs. Hydrostatic head method is used for leak tightness assessment after abrasion. 2. Classification of puncture resistance: Class 1 >5N; Class 2 >10N; Class 3 >50N; Class 4 >100N; Class 5 >150N; Class 6 >250N. 3. Classification of leak tightness after compression-folding (Schildknecht) flex cracking resistance: Class 1 >500cycles; Class 2 >1250cycles; Class 3 >3000cycles; Class 4 >8000cycles; Class 5 >20000cycles; Class 6 >50000cycles. Hydrostatic head method is used for leak tightness assessment after compression-folding (Schildknecht) flex cracking. Classification of leak tightness after compression-folding (Schildknecht) flex cracking resistance at -30°C: Class 1 >100cycles; Class 2 >200cycles; Class 3 >500cycles; Class 4 >1000cycles; Class 5 >2000cycles; Class 6 >4000cycles. Hydrostatic head method is used for leak tightness assessment after compression-folding (Schildknecht) flex cracking resistance at -30°C. 4. Classification of tensile strength: Class 1 >30N; Class 2 >60N; Class 3 >100N; Class 4 >250N; Class 5 >500N; Class 6 >1000N. 5. Classification of trapezoidal tear resistance: Class 1 >10N; Class 2 >20N; Class 3 >40N; Class 4 >60N; Class 5 >100N; Class 6 >150N. 6. Classification of seam strength: Class 1 >30N; Class 2 >50N; Class 3 >75N; Class 4 >125N; Class 5 >300N; Class 6 >500N.

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Repellency By Chemical Results (Type PB [6])

Chemikalie Chemical	Prüfmethode Test method	Resultat Result
Schwefelsäure (30%) 30% Sulphuric Acid	EN ISO 6530:2005	Klasse 3 Class 3
Natriumhydroxid (10%) 10% Sodium Hydroxide		Klasse 3 Class 3
Butan-1-ol		NT
o-Xylol o-xylene		NT

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Classification of repellency to liquids: Class 1 >70%; Class 2 >80%; Class 3 >90%. Classification is according to EN 14325:2018. NT= Not tested. Gown is unsuitable for use with Butan-1-ol, o-xylene, and organic chemicals.

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Resistance To Penetration Of Chemical Results (Type PB [6])

Chemikalie Chemical	Prüfmethode Test method	Resultat Result
Schwefelsäure (30%) 30% Sulphuric Acid	EN ISO 6530:2005	Klasse 3 Class 3
Natriumhydroxid (10%) 10% Sodium Hydroxide		Klasse 3 Class 3
Butan-1-ol		NT
o-Xylol o-xylene		NT

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Classification of resistance to penetration by liquids: Class 1 <10%; Class 2 <5%; Class 3 <1%. Classification is according to EN 14325:2018. NT= Not tested. Gown is unsuitable for use with o-xylene, Butan-1-ol and organic chemicals.

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1. Classification of Resistance to Penetration by Blood-Borne Pathogens (Hydrostatic pressure at which the material passes the test): Class 1 - 0kPa; Class 2 - 1.75kPa; Class 3 - 3.5kPa; Class 4 - 7kPa; Class 5 - 14kPa; Class 6 - 20kPa. 2. Classification of Resistance to Wet Microbial Penetration (Breakthrough time, t): Class 1-ts15 min; Class 2 - 15 min<ts30 min; Class 3 - 30 min<ts45 min; Class 4 - 45 min<ts60 min; Class 5 - 60 min<ts75 min; Class 6 - ts75 min. 3. Classification of Resistance to Liquid Aerosol Penetration (Penetration ratio (log)): Class 1 - 1<log≤3; Class 2 - 3<log≤5; Class 3 - log>5. 4. Classification of Resistance to Dry Microbial Penetration (Penetration (log cfu)): Class 1 - 2<log cfu≤3; Class 2 - 1<log cfu≤2; Class 3 - log cfu≤1. CfU = Colony forming units. Testing & Classification is based on EN 14126: 2003/AC:2004.

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The testing has been performed on the garment material. Seams and knitted cuffs have not been tested. Resistance to ignition has not been tested according to EN 14126.

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Information about the Medical Device

Category: Class I Medical Device according to Annex VIII of Regulation (EU) 2017/745

Intended purpose: The product is manufactured and indicated for the protection of others and protect against germ spreading in the nosocomial and nursing area.

Contra indication: The product must not be used in case of known allergies to the raw materials used.

Intended user: Medical and nursing staff.

Reporting of Incidents: All serious incidents related to the product must be reported immediately to the manufacturer and the competent authority.

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