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dach-germany.de | info@dach-germany.de

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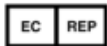
Jiangsu Medplus Non-woven Manufacturer Co., Ltd.

Address: No.217 East Wencheng Road, Economic Development Zone, Siyang,
223700 Jiangsu, P.R. China

Tel: 0086-527-80300123 Fax: 0086-021-51861120

Website: <http://www.cnmedplus.com>

The authorized representative of European Union:

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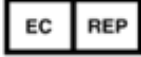
	CE certified by TÜV Rheinland		Batch code
	Do not re-use		Date of manufacture
	Sterilized using ethylene oxide		Manufacturer
	Catalogue number		European union representative
	Use-by date		Consult
	Do not use if package is damaged		Caution
	Keep away from sunlight		Keep dry
	Fragile, handle with care		Importer
	Medical device		Unique device identifier
	Nicht erneut sterilisieren		Vertriebspartner

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English

Product: Surgical drape and surgical drape (self-adhesive/adhesive)

Product name: IsoDrapeS und IsoDrapeS (selbstklebend/adhesive)

Product reference: REF 15S01BL und REF 15S02BL

Category: According to the classification criteria of MDD 93 /42 /EWG, Annex IX, this medical device is classified as Class Is. The product is used as a sterile product.

Intended purpose: Surgical Drapes are intended to be used as a protective patient covering, such as to isolate a site of surgical incision from microbial and other contamination. Disposable use.

Indication: It can be used to protect clinical medical staff during operation

Contraindication: none

Intended user: Medical staff

Use:

- Suitable for single use.
- Used drapes are considered contaminated
- Do not clean or disinfect
- Product are sterile

Incident notification: In the event of adverse reactions or serious events during the use of the device, contact the manufacturer, importer on the device label or contact the local authority.

Storage:

1. The storage environment shall be kept non-corrosive gases, dry, avoiding sunlight, well-ventilated conditions, and clean.
2. The device shall be taken care during transportation and handling.

Storage period: If the storage conditions are observed the product has a storage period of 5 years (see labelling on the product).

Sterilization: Ethylene oxide sterilized

Disposal: Single-use only, must follow the sterile operation process; After use, should be treated as biohazard.

Precautions:

Please check the package before use, if the package should be damaged or accidentally opened, do not use.

Check the label of the outer package (production date and validity period) and use within the validity period of sterilisation.

Dressing instructions:

Open the package, then put the surgical drapes on the bed or cover the patient. For the surgical drapes with adhesive tape, we should make the tape stick to patient skin tightly in case of any dropping.

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DACH Schutzbekleidung GmbH & Co. KG
Shanghai International Holding Corp. GmbH