Deutsch

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Product name: MediPro Loop

Product reference: 2108/B, 2108/G

INSTRUCTION FOR USE: Intended purpose: Medical face masks are manufactured and indicated for foreign protection and protect the counterpart from exposure to potentially infectious droplets from the person wearing the medical face mask.

Intended user: Medical staff and nursing personnel as well as end users. **Contra indication:** Masks may be unsuitable for users who have received a corresponding indication from their doctor, e.g., due to a pre-existing restriction or illness of the respiratory tract. The mask contains metal parts in the nose bridge, therefore it is not suitable for use in strong magnetic fields or potentially explosive areas. **Fitting Instructions:** 1. Wash your hands with water and soap or sanitize your hands with an alcohol-based hand sanitizer before touching the mask. 2. Hold the mask in front of your face: the colored side should face outwards and the edge with the nose wire should be at the top. 3. a) Mask with ear loops: Grab the ear loops and pull them behind your ears. b) Mask with ties: Grab the ties at the top of the mask and tie them at the back of your head. Bend the nose wire with both hands around your nose to achieve a good fit on your face. 4. While holding the mask with one hand, pull the bottom part of the mask under your chin to stretch out the folds of the mask. Mask with ties: Grab the ties at the bottom of the mask and tie them at the nape of your neck. The mask should now completely cover your nose, mouth, and jaw. 5. If you wear glasses, put them over the mask. **IMPORTANT:** Do not touch the contaminated outside of the mask during or after use.

Remove the mask by pulling the ear loops from the ears. Wash and disinfect your hands after use. If the mask becomes damp, replace it immediately. **Warning:** Inspect the mask, including all components, for damage before use. If any damage is found, DO NOT use the mask. **Storage and Shelf Life:** Store in a dry environment and protected from direct sunlight in the original packaging, see pictograms on the packaging. Under the storage conditions, the product has a shelf life of 5 years (see labeling on the product). **Disposal:** Dispose of the mask immediately and properly, i.e., in a closed container. National regulations and laws must be followed. **Categorization:** Medical device Class I according to Regulation (EU) 2017/745. Meets the requirements of standard EN 14683:2019+AC:2019. **Reporting Incidents:** All serious incidents related to the product must be reported immediately to the manufacturer and the competent authority. If you have any further questions, you can contact us by phone.

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DACH Schutzbekleidung GmbH & Co. KG