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Preliminary remarks on the Instructions for use

Our instructions for use contain all required information. They enable identification of the product and manufacturer, and safe and adequate application. They are available in all relevant languages. These full instructions for use can be viewed as an independent document. Following included are the text passages of the instructions for use for ubiQ²® and ubiQ³® in English, German, French, Italian und Spanish.

The instruction for use is directly printed on the product dispenser box, which also contains information about the notified body and the manufacturer address. For the complete information, see 27x | ubiQ® | Packaging & Labelling .

ubiQ²® and ubiQ³®:Text of the instructions for use in different languages

☰ The following texts show the written part of the instructions for use, which are applicable for all masks of the ubiQ® series, since they contain all relevant information for both protection classes; FFP2 and FFP3. The product references **272/M** **272/L** **278/M** **278/L** **273/M** and **273/L** can be supplied with these instructions for use. The information about the Notified Body and manufacturer Address can be found on the product packaging, where these instructions are printed on.

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English: Filtering half masks - INSTRUCTIONS FOR USE

IMPORTANT:

The ubiQ® filtering half masks are Personal Protective Equipment Category III in compliance with Regulation (EU) 2016/425 against solid and liquid aerosols. Type tested according to EN 149:2001+A1:2009. Protection category FFP2 or FFP3 as marked on the mask.

APPLICATIONS:

FFP2: Filtering half masks of category FFP2 protect against solid and liquid particles of all fine dusts up to 10 times WEL, [i.e. APF=10]. FFP3: Filtering half masks of category FFP3 protect against solid and liquid particles of all fine dusts up to 20 times WEL, [i.e. APF=20].

CHECK BEFORE USE:

Before use, make sure that the filtering half mask, including all components, has no holes, cracks or other damage. If you discover any damage, DO NOT USE THE MASK. Use a new one.

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
National regulations must be observed, e.g. DGUV regulation 112-190 (BGR 190) "Use of respiratory protective equipment", BGI 504-26 "Selection criteria for special occupational medical precautions in accordance with the employers' liability insurance association principle, G26 Respiratory protective equipment".

FITTING INSTRUCTIONS:

1. Unfold the mask into a bowl shape.
2. Place the lower half under the chin. Pull the elastic band behind the head.
3. Place lower band around back of neck.
4. Place the upper band on the back of the head above the ears and adjust so that it applies even pressure all over.
5. Use both hands to press the nose clip against the nose to seal the area around the nose.
6. Tightness test: Grasp the mask with both hands and exhale forcefully. If there are leaks, adjust the mask slightly to fit the shape of the face.

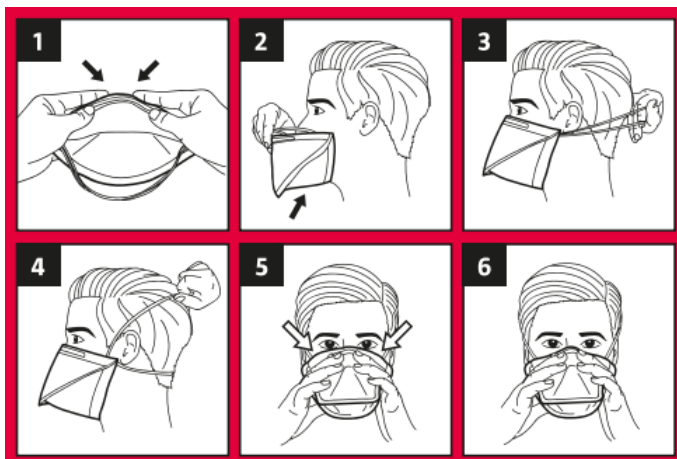***NOTE:** Replace the mask immediately if breathing becomes difficult or if the mask is damaged or deformed in such a way that an optimal seal can no longer be guaranteed. Use mask only in accordance with these fitting instructions.


WARNINGS:

1. Failure to follow the instructions given here or restrictions on use of this product may reduce the effectiveness of this mask and result in nausea or even death.
2. A carefully selected mask is essential to protect your health. Before using this mask, consult an appropriate safety organisation to determine if this mask meets the requirements of the intended use.
3. Filtering half masks are dependent on the ambient atmosphere. Use only in adequately ventilated environments. Do not use if the atmosphere contains less than 17% oxygen.
4. Respirators must not be used if the concentration of harmful substances poses an immediate threat to health or life.
5. Do not use in an explosive atmosphere.
6. Leave the work area immediately,
 - a. if breathing becomes difficult,
 - b. dizziness or exhaustion occurs.
7. Facial hair and special characteristics of the face shape may reduce the effectiveness of this mask.
8. Do not make any changes or modifications to the mask.

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9. Intended for single use.
No maintenance required.
Dispose of properly after use.
10. Store masks in packaging away from direct sunlight until ready to use.

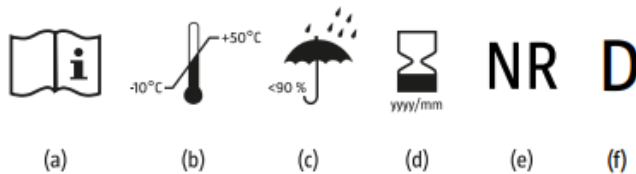MEANING OF THE ICONS/SYMBOLS:

1. Meaning

- a. See user information.
- b. Temperature range during storage.
- c. Maximum humidity level during storage.
- d. End of shelf life. yyyy = year; mm = month.
- e. NR: suitable for single use only
(maximum wearing time of 8 hours).
Do not reuse.
- f. D: this respirator passes the optional Dolomite clogging test for more comfortable breathing.

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