24xxx | Klassik | Gebrauchsanweisung / Instructions for Use / Instructions d'utilisation

Respirators Cla... use CE2797.pdf

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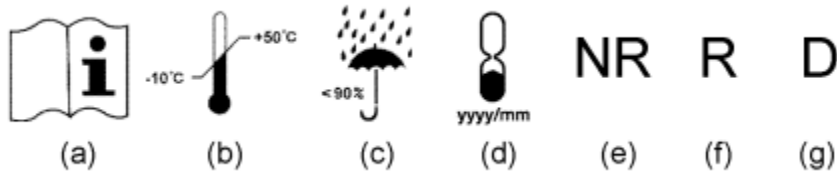
The following texts constitute the written part of the instructions for use for all masks in the Classic series, as it contains all the relevant information for protection classes FFP1, FFP2 and FFP3. A printed copy of these instructions is included with each primary packaging. The product references **242S** **242SV** **243SV** can be supplied with these instructions for use. The information on the Notified Body and the manufacturer's address can also be found on the product packaging.

<EN> ENGLISH: INSTRUCTIONS FOR USE

IMPORTANT: PPE Regulation (EU) 2016/425, EN 149:2001+A1:2009 respirators against solid and liquid aerosols are classified according to one of three categories - FFP1, FFP2, and FFP3. Before reading the following information, please check the category to which the respirator belongs - this is indicated on the packaging and on the respirator.

APPLICATIONS: • **FFP1:** protection against non-toxic solid and liquid aerosols (e.g. oil-mists) in concentrations up to 4 x MAC/OEL/TLV, [i.e. NPF=4] or 4 x WEL, [i.e. APF=4]. • **FFP2:** protection against non-toxic and low-to-average toxicity solid and liquid aerosols (e.g. oil-mists) in concentrations up to 12 x MAC/OEL/TLV, [i.e. NPF=12] or 10 x WEL, [i.e. APF=10]. • **FFP3:** protection against non-toxic, low-to-average toxicity and high toxicity solid and liquid aerosols (e.g. oil-mists) in concentrations up to 50 x MAC/OEL/TLV, [i.e. NPF=50] or 20 x WEL, [i.e. APF=20]. • **DO NOT USE** this respirator against vapour/gas in concentrations more than 1 x MAC/OEL/TLV, [i.e. NPF=1] or 1 x WEL, [i.e. APF=1]. ***NOTE :** MAC = Maximum Admissible Concentration. OEL = Occupational Exposure Limit. TLV = Threshold Limit Value. NPF = Nominal Protection Factor. WEL = Workplace Exposure Limit. APF = Assigned Protection Factor.

CHECK PRIOR TO USE: Inspect respirator, including any component parts, for holes, tears or damage prior to use. Should any damage be identified, **DO NOT USE IT.** Get a new one.

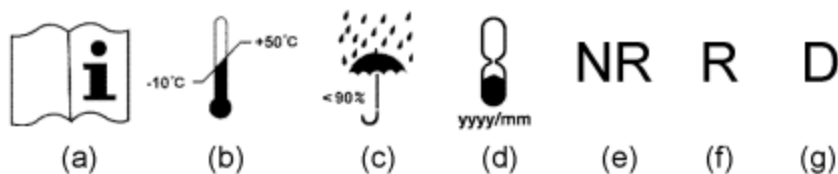
FITTING INSTRUCTIONS : **1.** Hold the respirator in hand with the nosepiece at your fingertips. Allow headbands to hang freely below hand. **2.** Cup the respirator firmly against your face with the nosepiece on the bridge of your nose. **3.** Stretch and pull the lower headband over the head and position below your ears. Stretch and pull the top headband on the back of your head above your ears. ***Respirators with adjustable headbands only :** while maintaining the position of the respirator, adjust tension by pulling tab of each headband. (Tension may be decreased by pushing out the back of the buckle.) **4.** Press soft nosepiece to conform snugly around the nose. **5.** To test fit, cup both hands and inhale sharply. A negative pressure should be felt inside respirator. If air flows around your nose, tighten the nosepiece. If air leaks around the edges, reposition the headband for a better fit. ***NOTE :** Change respirator immediately if breathing becomes difficult or respirator becomes damaged or distorted, or a proper face fit can not be maintained. Respiratory protection is only effective if it is correctly selected, fitted, and worn throughout the time when the wearer is exposed to hazards.

WARNING : **1.** Failure to follow all instructions and limitations on the use of this product could reduce the effectiveness of respirator and result in sickness or death. **2.** A properly selected respirator is essential to protect your health. Before using this respirator, consult an Industrial Hygienist or Occupational Safety Professional to determine the suitability for your intended use. **3.** This product does not supply oxygen. Use only in adequately ventilated areas containing sufficient oxygen to support life. Do not use this respirator when oxygen concentration is less than 19%. **4.** Do not use when concentrations of contaminants are immediately dangerous to health or life. **5.** Do not use in explosive atmospheres. **6.** Leave work area immediately and return to fresh air if (a) breathing becomes difficult, or (b) dizziness or other distress occurs. **7.** Facial hairs or beards and certain facial characteristics may reduce the effectiveness of this respirator. **8.** Never alter or modify this respirator in any way. **9.** For respirators bearing the symbol "NR" : Only for single shift use. No maintenance necessary. Discard the used respirator after single shift use. **10.** For respirators bearing the symbol "R" : The respirator may be re-used by a single individual wearer. If you decide to reuse this respirator, please follow the below cleaning instructions after each use : **(1)** Cleaning is recommended after each use. **(2)** Clean only the full facial seal by employing a clean cloth with warm water not to exceed 50°C, if necessary, with a neutral cleaning agent. **(3)** Do not use cleaning products

containing lanolin or other oils. **(4)** Visually inspect the exhalation valve for damage. If damage or wear is evident, discard the respirator. **(5)** Ensure that the respirator is dry before re-use. **11.** Keep un-used respirators in the display box away from direct sunlight until use.

Manufacturer/Importer : Refer to packaging for company name and address. **EU Declaration of Conformity :** Refer to packaging for internet download address.

MEANING OF PICTOGRAMS/SYMBOLS : (a) See user information. (b) Temperature range of storage condition. (c) Maximum relative humidity of storage condition. (d) End of shelf life. yyyy = Year; mm = Month. (e) NR : this respirator is limited to single shift use only (with a maximum of 8 hours) and shall not be re-used. (f) R : this respirator is re-usable (more than one shift). (g) D : this respirator passes the optional Dolomite clogging test for more comfortable breathing.

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