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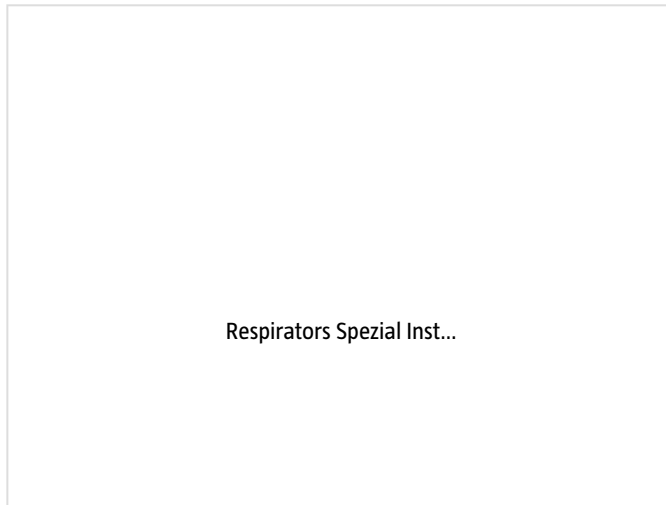
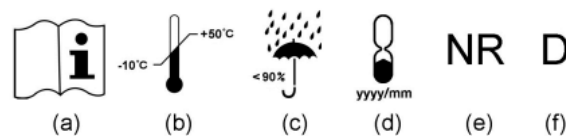
The following texts constitute the written part of the instructions for use for all masks in the Special series. A printed copy of these instructions is included with each primary packaging. The product references **242D** **242DAV** can be supplied with these instructions for use. The information on the Notified Body and the manufacturer's address can also be found on the product packaging.

### <EN> ENGLISH: **INSTRUCTIONS FOR USE**

**IMPORTANT:** PPE Regulation (EU) 2016/425, EN 149:2001+A1:2009 respirators against solid and liquid aerosols are classified according to one of three categories - FFP1, FFP2, and FFP3. Before reading the following information, please check the category to which the respirator belongs - this is indicated on the packaging and on the respirator.

**APPLICATIONS:** • **FFP1**: protection against non-toxic solid and liquid aerosols (e.g. oil-mists) in concentrations up to 4 x MAC/OEL/TLV, [i.e. NPF=4] or 4 x WEL, [i.e. APF=4]. • **FFP2**: protection against non-toxic and low-toaverage toxicity solid and liquid aerosols (e.g. oil-mists) in concentrations up to 12 x MAC/OEL/TLV, [i.e. NPF=12] or 10 x WEL, [i.e. APF=10]. • **FFP3**: protection against non-toxic, low-to-average toxicity and high toxicity solid and liquid aerosols (e.g. oil-mists) in concentrations up to 50 x MAC/OEL/TLV, [i.e. NPF=50] or 20 x WEL, [i.e. APF=20]. • **DO NOT USE** this respirator against vapour/gas in concentrations more than 1 x MAC/OEL/TLV, [i.e. NPF=1] or 1 x WEL, [i.e. APF=1]. \***NOTE**: MAC = Maximum Admissible Concentration. OEL = Occupational Exposure Limit. TLV = Threshold Limit Value. NPF = Nominal Protection Factor. WEL = Workplace Exposure Limit. APF = Assigned Protection Factor.

**CHECK PRIOR TO USE:** Inspect respirator, including any component parts, for holes, tears or damage prior to use. Should any damage be identified, **DO NOT USE IT.** Get a new one.

**FITTING INSTRUCTIONS:** **1.** Hold mask in position over nose and mouth, with retaining straps around back of head. Pull lower portion of strap over head and fit around neck. **2.** Stretch and pull the upper strap over your head on the back of your head above your ears. **3.** Adjust the tension in the upper and lower portions of the continuous loop strap, to ensure a

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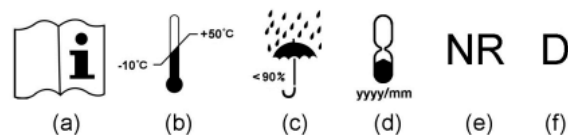
positive face seal. **4.** Tension may be increased by pulling strap at either side of buckle to obtain a tighter facial fit. Tension may be decreased by pushing out the back of the buckle to obtain a looser facial fit. **5.** To test fit, cup both hands over the respirator and inhale sharply. A negative pressure should be felt inside respirator. If air flows around your nose, reposition the respirator. If air leaks around the edges, repeat point 3 and 4. **6.** During work break, you may take off the lower portion of strap. Let the respirator hang around the neck. **\*NOTE :** Change respirator immediately if breathing becomes difficult or respirator becomes damaged or distorted, or a proper face fit can not be maintained. Respiratory protection is only effective if it is correctly selected, fitted, and worn throughout the time when the wearer is exposed to hazards.

**WARNING : 1.** Failure to follow all instructions and limitations on the use of this product could reduce the effectiveness of respirator and result in sickness or death. **2.** A properly selected respirator is essential to protect your health. Before using this respirator, consult an Industrial Hygienist or Occupational Safety Professional to determine the suitability for your intended use. **3.** This product does not supply oxygen. Use only in adequately ventilated areas containing sufficient oxygen to support life. Do not use this respirator when oxygen concentration is less than 19%. **4.** Do not use when concentrations of contaminants are immediately dangerous to health or life. **5.** Do not use in explosive atmospheres. **6.** Leave work area immediately and return to fresh air if (a) breathing becomes difficult, or (b) dizziness or other distress occurs. **7.** Facial hairs or beards and certain facial characteristics may reduce the effectiveness of this respirator. **8.** Never alter or modify this respirator in any way. **9.** Only for single shift use. No maintenance necessary. Discard the used respirator after single shift use. **10.** Keep un-used respirators in the display box away from direct sunlight until use.

**Manufacturer/Importer :** Refer to packaging for company name and address. **EU Declaration of Conformity :** Refer to packaging for internet download address.

**MEANING OF PICTOGRAMS/SYMBOLS :** (a) See user information. (b) Temperature range of storage condition. (c) Maximum relative humidity of storage condition. (d) End of shelf life. yyyy = Year; mm = Month. (e) NR : this respirator is limited to single shift use only (with a maximum of 8 hours) and shall not be re-used. (f) D : this respirator passes the optional Dolomite clogging test for more comfortable breathing.

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