411 | EcoCoat pro | Gebrauchsanweisung / Instructions for Use

Produkt / Product

Produktname / Product name: EcoCoat pro

Produktreferenz / product reference: 411

Kategorisierung/ Category

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Category I Personal Protective Equipment according to Annex I of Regulation (EU) 2016/425 on personal protective equipment and Class I medical devices according to Annex VIII of Regulation (EU) 2017/745.

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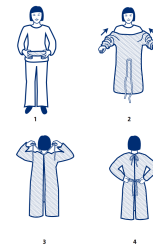
PSA: The gown is designed to be worn over other (work) clothing and serves to protect the wearer from non-toxic dirt and to prevent contamination.

Medical Device: The product is manufactured and indicated for the protection of others and protect against germ spreading in the nosocomial and nursing area.

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1. *Take the gown out of the bag with clean hands.*
2. *Slip into sleeves and pull gown over shoulders.*
3. *Close the gown on the back of the neck.*
4. *Close the gown at the back at the waist area.*

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Check product for damage before use. If damaged, use new product.

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In original packaging, in a dry and dark place, without direct sunlight, at approx. -10°C to 40°C room temperature. Storage period of 10 years from production under compliance with storage conditions.

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When used as intended, the product can be disposed of in household waste and landfills and thermally recycled without releasing toxic substances. Nationally applicable regulations and laws must be followed.

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Not required, as it is a disposable product.

PSA Schutzklassen und Verwendungsgrenzen / PPE Classes of protection and corresponding limits of use

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The product has no protection class. Does not protect against substances hazardous to health or harmful biological agents.

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This product complies with the harmonized standards according to Regulation (EU) 2016/425.

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The product must not be used in case of known allergies to the raw materials used.

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All serious incidents related to the product must be reported immediately to the manufacturer and the competent authority.

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If you have any further questions, you can contact us at the following telephone number:

 +49 7222 159 89-0

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DACH Schutzbekleidung GmbH Co. KG