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i Following included are the text passages of the instructions for use for Neolution®Air Loop in English, German and French. The instruction for use is directly printed on the product dispenser box, which also contains information about the notified body and the manufacturer address. In case of the packaging variant in Bags, the instructions are printed and added to every smallest saleable unit, which is the shipping carton. For the complete information, see 25x | Neolution®Air | Packaging & Labelling.

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ENGLISH: Filtering half masks - INSTRUCTIONS FOR USE

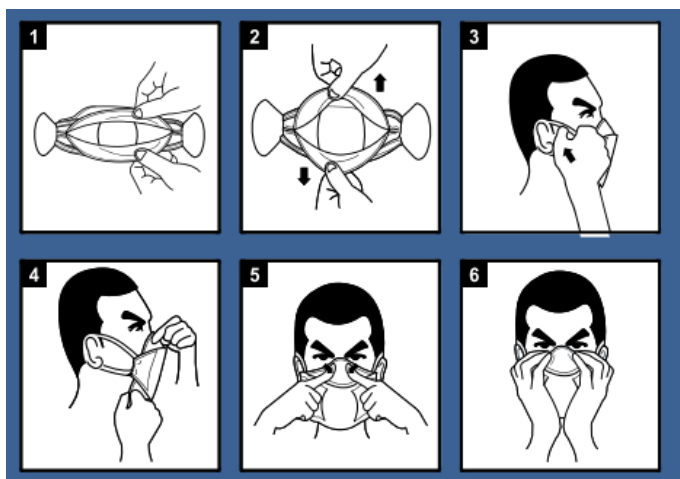
IMPORTANT: The Neolution®Air Loop filtering half masks are Personal Protective Equipment Category III in compliance with Regulation (EU) 2016/425 against solid and liquid aerosols. Type tested according to EN 149:2001+A1:2009. Protection category FFP2 or FFP3 as marked on the mask.

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APPLICATIONS: FFP2: Filtering half masks of category FFP2 protect against solid and liquid particles of all fine dusts up to 10 times WEL, [i.e. APF=10]. FFP3: Filtering half masks of category FFP3 protect against solid and liquid particles of all fine dusts up to 20 times WEL, [i.e. APF=20].


CHECK BEFORE USE: Before use, make sure that the filtrating half mask, including all components, has no holes, cracks or other damage. If you discover any damage, DO NOT USE THE MASK. Use a new one. National regulations must be observed, e.g. DGUV regulation 112-190 (BGR 190) "Use of respiratory protective equipment", BGI 504-26 "Selection criteria for special occupational medical precautions in accordance with the employers' liability insurance association principle, G26 Respiratory protective equipment".

FITTING INSTRUCTIONS: 1. Separate top and bottom panels at the back of the mask to form a cup shape. 2. Ensure both panels are fully unfolded. 3. Cup the bottom panel under your chin and pull both elastics behind your ears. 4. Adjust top and bottom panels for a comfortable fit and ensure panels are not folded inwards. 5. Using both hands, press nose clip around nose to the shape of the nose for a good facial seal. 6. To test the fit, cup both hands over the respirator and inhale sharply. A negative pressure should be felt inside respirator. If air flows around your nose, tighten the nosepiece. If air leaks around the edges, reposition the ear loops for better fit. *NOTE: Change respirator immediately if breathing becomes difficult or respirator becomes damaged or deformed in such a way that an optimal seal can no longer be guaranteed. Use mask only in accordance with these fitting instructions.

WARNINGS: 1. Failure to follow the instructions given here or restrictions on use of this product may reduce the effectiveness of this mask and result in nausea or even death. 2. A carefully selected mask is essential to protect your health. Before using this mask, consult an appropriate safety organisation to determine if this mask meets the requirements of the intended use. 3. Filtering half masks are dependent on the ambient atmosphere. Use only in adequately ventilated environments. Do not use if the atmosphere contains less than 19 % oxygen. 4. Respirators must not be used if the concentration of harmful substances poses an immediate threat to health or life. 5. Do not use in an explosive atmosphere. 6. Leave the work area immediately, (a) if breathing becomes difficult, (b) dizziness or exhaustion occurs. 7. Facial hair and special characteristics of the face shape may reduce the effectiveness of this mask. 8. Do not make any changes or modifications to the mask. 9. Intended for single use. No maintenance required. Dispose of properly after use. 10. Store masks in packaging away from direct sunlight until ready to use.

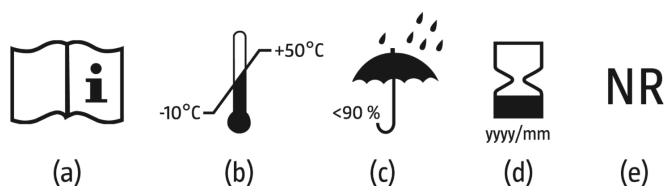MEANING OF THE ICONS/SYMBOLS: (a) See user information. (b) Temperature range during storage. (c) Maximum humidity level during storage. (d) End of shelf life. yyyy = year; mm = month. (e) NR: suitable for single use only (maximum wearing time of 8 hours). Do not reuse.

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Marking and performance recorded in technical tests applicable to this PPE:

neolution Air Loop

REF 253 | FFP2 NR | EN 149:2001+A1:2009

YYYY/MM

CE 0161

Meaning marked: manufacturer's brand -

PPE reference | Classification: FFP2 NR | Applicable Standard - expiration date - CE-marking + Notified Body of production control: CE 0161

Levels or classes of protection offered by PPE intended for respiratory protection: (1) Maximum penetration of the filter material (Paraffin Aerosols and Sodium Chloride) (2) Breathing resistance (3) Inward Leakage

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