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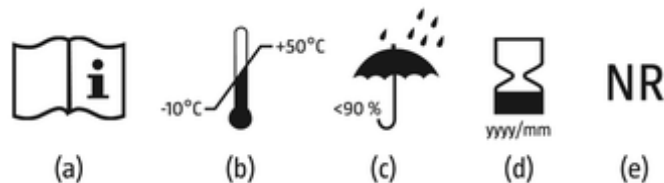
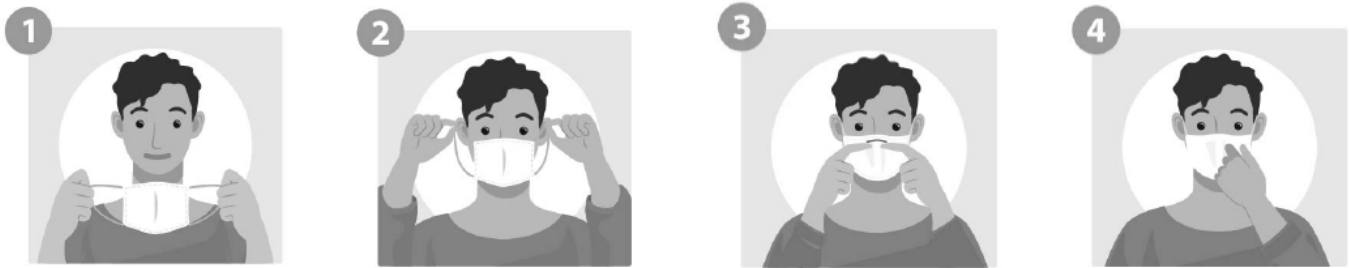
ENGLISH: Filtering half masks - INSTRUCTIONS FOR USE

IMPORTANT: The C-Mask filtering half mask is Personal Protective Equipment Category III in compliance with Regulation (EU) 2016/425 against solid and liquid aerosols. Type tested according to EN 149:2001+A1:2009. Protection category FFP2 NR as marked on the mask.

APPLICATIONS: FFP2: Filtering half masks of category FFP2 protect against solid and liquid particles of all fine dusts up to 10 times WEL, [i.e. APF=10].

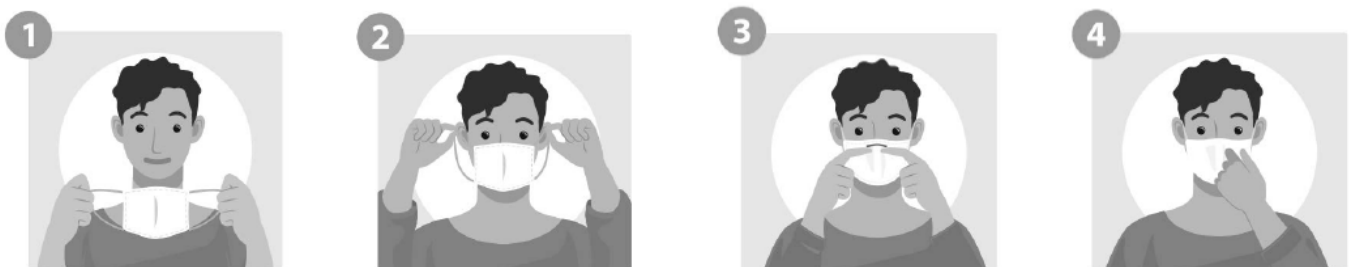
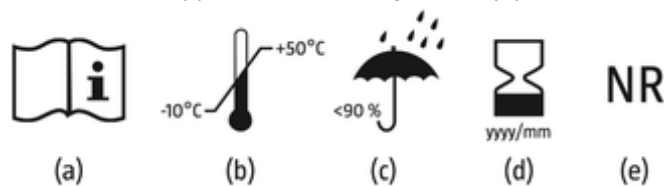
CHECK BEFORE USE: Inspect respirator, including any component parts, for holes, tears or damage prior to use. Should any damage be identified, **DO NOT USE IT**. Get a new one. National regulations must be observed, e.g. DGUV regulation 112-190 (BGR 190) "Use of respiratory protective equipment", BGI 504-26 "Selection criteria for special occupational medical precautions in accordance with the employers' liability insurance association principle, G26 Respiratory protective equipment".

FITTING INSTRUCTIONS: **1.** Fold the mask open to form a cup shape. Place fingers inside the respirator and bend the nose clip around fingers to form nose shape. **2.** Holding mask in position over nose and mouth, pull both elastics behind your ears. **3.** Using both hands, form nose clip around nose to ensure a positive face seal. **4.** To test the fit, cup both hands over the respirator and inhale sharply. A negative pressure should be felt inside respirator. If air flows around your nose, tighten the nosepiece. If air leaks around the edges, reposition the ear loops for better fit. ***NOTE:** Change respirator immediately if breathing becomes difficult or respirator becomes damaged or deformed in such a way that an optimal seal can no longer be guaranteed. . Respiratory protection is only effective if it is correctly selected, fitted, and worn throughout the time when the wearer is exposed to hazards.

WARNINGS: **1.** Failure to follow the instructions given here or restrictions on use of this product may reduce the effectiveness of this mask and result in sickness or even death. **2.** A properly selected respirator is essential to protect your health. Before using this respirator, consult an Industrial Hygienist or Occupational Safety Professional to determine the suitability for your intended use. **3.** This product does not supply oxygen. Use only in adequately ventilated areas containing sufficient oxygen to support life. Do not use this respirator when oxygen concentration is less than 19%. **4.** Do not use when concentrations of contaminants are immediately dangerous to health or life. **5.** Do not use in explosive atmospheres. **6.** Leave work area immediately and return to fresh air if (a) breathing becomes difficult, or (b) dizziness or other distress occurs. **7.** Facial hairs or beards and certain facial characteristics may reduce the effectiveness of this respirator. **8.** Never alter or modify this respirator in any way. **9.** Only for single shift use. No maintenance necessary. Discard the used respirator after single shift use. **10.** Keep unused respirators in the display box away from direct sunlight until use.

Manufacturer: Refer to packaging for company name and address. **EU Declaration of Conformity:** Refer to packaging for internet download address.

MEANING OF THE ICONS/SYMBOLS: (a) See user information. (b) Temperature range during storage. (c) Maximum humidity level during storage. (d) End of shelf life. yyyy = year; mm = month. (e) NR: suitable for single use only (maximum wearing time of 8 hours). Do not reuse.

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