<**EN**> **ENGLISH: Filtering face-piece respirators**

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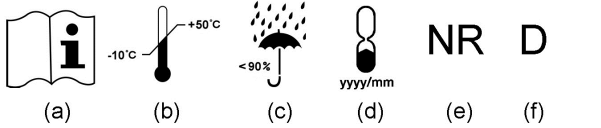
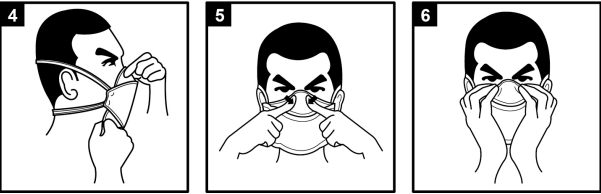
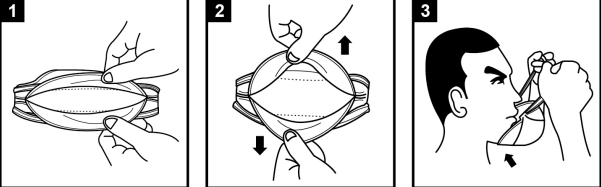
Say Building, John M. Keynesplein 9, 1066 EP Amsterdam, Netherlands

<**EN**> **ENGLISH : INSTRUCTIONS FOR USE**

IMPORTANT : PPE Regulation (EU) 2016/425, EN 149:2001+A1:2009 respirators against solid and liquid aerosols are classified according to one of three categories - FFP1, FFP2, and FFP3. Before reading the following information, please check the category to which the respirator belongs - this is indicated on the packaging and on the respirator.

APPLICATIONS : • **FFP1** : protection against non-toxic solid and liquid aerosols (e.g. oil-mists) in concentrations up to 4 x MAC/OEL/TLV, [i.e. NPF=4] or 4 x WEL, [i.e. APF=4]. • **FFP2** : protection against non-toxic and low-to-average toxicity solid and liquid aerosols (e.g. oil-mists) in concentrations up to 12 x MAC/OEL/TLV, [i.e. NPF=12] or 10 x WEL, [i.e. APF=10]. • **FFP3** : protection against non-toxic, low-to-average toxicity and high toxicity solid and liquid aerosols (e.g. oil-mists) in concentrations up to 50 x MAC/OEL/TLV, [i.e. NPF=50] or 20 x WEL, [i.e. APF=20]. • **DO NOT USE** this respirator against vapour/gas in concentrations more than 1 x MAC/OEL/TLV, [i.e. NPF=1] or 1 x WEL, [i.e. APF=1]. ***NOTE** : MAC = Maximum Admissible Concentration. OEL = Occupational Exposure Limit. TLV = Threshold Limit Value. NPF = Nominal Protection Factor. WEL = Workplace Exposure Limit. APF = Assigned Protection Factor.

CHECK PRIOR TO USE : Inspect respirator, including any component parts, for holes, tears or damage prior to use. Should any damage be identified, **DO NOT USE IT**. Get a new one.

FITTING INSTRUCTIONS :
1. With reverse side up and separate top and bottom panels to form a cup shape.
2. Ensure both panels are fully unfolded.
3. Cup the bottom panel under your chin and take both straps over head.
4. Locate the lower strap around the neck. Locate the upper strap above the ears. Pull and adjust top and bottom panels for a comfortable fit and ensure panels are not folded in.
5. Using both hands, press nose clip around nose to the shape of the lower part of the nose for a good facial seal.
6. To test fit, cup both hands over the respirator and inhale sharply. A negative pressure should be felt inside respirator. If air flows around your nose, tighten the nosepiece. If air leaks around the edges, reposition the headband for better fit.
***NOTE** : Change respirator immediately if breathing becomes difficult or respirator becomes damaged or distorted, or a proper face fit can not be maintained. Respiratory protection is only effective if it is correctly selected, fitted, and worn throughout the time when the wearer is exposed to hazards.

WARNING :
1. Failure to follow all instructions and limitations on the use of this product could reduce the effectiveness of respirator and result in sickness or death.
2. A properly selected respirator is essential to protect your health. Before using this respirator, consult an Industrial Hygienist or Occupational Safety Professional to determine the suitability for your intended use.
3. This product does not supply oxygen. Use only in adequately ventilated areas containing sufficient oxygen to support life. Do not use this respirator when oxygen concentration is less than 19%.
4. Do not use when concentrations of contaminants are immediately dangerous to health or life.
5. Do not use in explosive atmospheres.
6. Leave work area immediately and return to fresh air if (a) breathing becomes difficult, or (b) dizziness or other distress occurs.
7. Facial hairs or beards and certain facial characteristics may reduce the effectiveness of this respirator.
8. Never alter

or modify this respirator in any way.
9. Only for single shift use. No maintenance necessary. Discard the used respirator after single shift use.
10. Keep un-used respirators in the display box away from direct sunlight until use.

Manufacturer/Importer : Refer to packaging for company name and address.
EU Declaration of Conformity : Refer to packaging for internet download address.

MEANING OF PICTOGRAMS/SYMBOLS : (a) See user information. (b) Temperature range of storage condition. (c) Maximum relative humidity of storage condition. (d) End of shelf life. yyyy = Year; mm = Month. (e) NR : this respirator is limited to single shift use only (with a maximum of 8 hours) and shall not be re-used. (f) D : this respirator passes the optional Dolomite clogging test for more comfortable breathing.

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